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
IEC (International Electrotechnique Commission)

IEC 60601 Medical Electrical Equipment

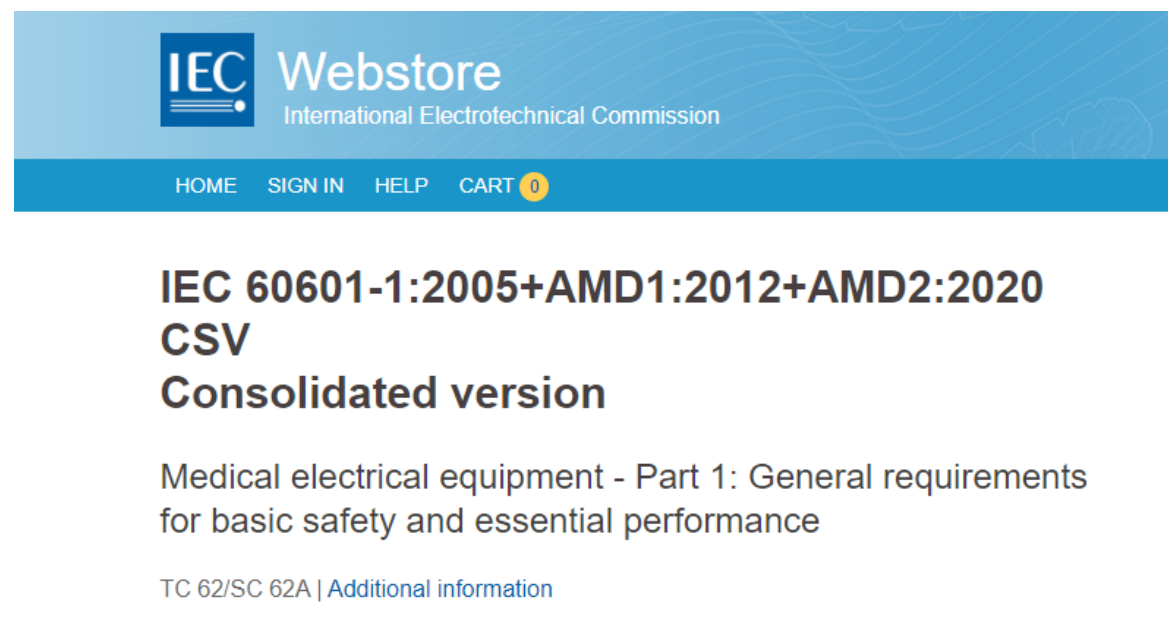IEC 60601-1 Medical Electrical Equipment Part 1:
General requirements for basic safety and
essential performance

IEC Webstore
International Electrotechnical Commission

HOME SIGN IN HELP CART 0

**IEC 60601-1:2005+AMD1:2012+AMD2:2020
CSV
Consolidated version**

Medical electrical equipment - Part 1: General requirements
for basic safety and essential performance

[TC 62/SC 62A | Additional information](#)

IEC 62353 Medical electrical equipment -
Recurrent test and test after repair of medical
electrical equipment

IEC Webstore
International Electrotechnical Commission

HOME SIGN IN HELP CART 0

IEC 62353:2014

Medical electrical equipment - Recurrent test and test after
repair of medical electrical equipment

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ECRI : Emergency Care Research Institute

ECRI is an independent, nonprofit organisation improving the safety, quality, and cost-effectiveness of care across all healthcare settings worldwide. We empower healthcare leaders to make the best decisions to deliver safe, effective, and efficient care.

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ECRI : Emergency Care Research Institute

ECRI About ECRI News Events Contact Us Supot

My ECRI Browse Topics **Memberships & Services**

Your browsing history

IPM Procedures

These are generic IPM procedures that have been developed and approved by ECRI Institute. You may download the MS Word documents and customize them. Some of the procedures have a dedicated inspection form. Use the General Devices form as a template to develop forms for other procedures.

The procedures contains tasks for Acceptance and Scheduled inspections. Tasks not labeled [Acceptance] or [Scheduled] apply to both types of inspection.

The following document summarizes the suggested inspection interval and estimated inspection time for each procedure: [IPM Procedure Intervals and Times](#).

Cyber indicates IPM procedures with specific cybersecurity tasks.

Cyber Ambulatory Infusion Pumps

Cyber Anesthesia Units

CONTACT US

I need help with...

General Questions
clientservices@ecri.org
 (610) 825-6000 ext. 5891

[Request More Information >](#)
[ECRI Directory >](#)
[Report a Device Problem >](#)

BiomedicalBenchmark™ THE TECHNOLOGY SUPPORT SYSTEM

ECRI INSTITUTE IPM PROCEDURE INSPECTION INTERVALS AND TIMES

| Procedure Name | Interval (mo) | Time (hr) |
|--|---------------|------------------------|
| Ambulatory Infusion Pumps | 12 | 0.7 |
| Anesthesia Units | 6 | 2.4 |
| Anesthesia Vaporizers | 6 | 0.7 |
| Apnea Monitors | 12 | 0.3 |
| Argon-Enhanced Coagulation Units | 12 | 1.0 |
| Argon Surgical Lasers | 6 | 1.0 |
| Autotransfusion Units | 6 | 0.8 |
| Blood and Solution Warmers | 12 | 0.8 |
| Breast Pumps | 12 | 0.2 |
| Bulk Steam Sterilizers | 3 | 6.0 |
| Capnometers/Multiple Medical Gas Monitors | 12 | 0.8 |
| Carbon Dioxide Surgical Lasers | 6 | 1.0 |
| Centrifuges | 12 | 0.5 |
| Circulating Fluid Pumps | 12 | 0.4 |
| Computed Radiography Systems | 1 | 1.0 |
| Continuous/Bi-level Positive Airway Pressure Units | 12 | 0.5 |
| Contrast Media Injectors | 12 | 0.6 |
| Cryosurgical Units | 12 | 0.6 |
| Defibrillators and Noninvasive Pacemakers | 6 | 0.4/defib, 0.6 both |
| Diagnostic Imaging Workstations | 6 | 0.1/monitor |
| Electric Beds | 12/24 | 0.3 |
| Electrical Receptacles | 12 | 100 duplex recepts/hr* |
| Electrocardiographs | 12 | 0.5 |
| Electrosurgical Units | 12 | 0.6 |
| Enteral Feeding Pumps | 12 | 0.4 |

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High Risk

Life-support, key resuscitation, critical monitoring, energy emitting, and other devices whose failure or misuse is reasonably likely to seriously injure patients or staff.

Medium Risk

Devices, including many diagnostic instruments, whose misuse, failure or absence (e.g., out of service with no replacement available) would have a significant impact on patient care but would not be likely to cause direct serious injury.

Low Risk

Devices whose failure or misuse is unlikely to result in serious consequences.

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CONDITION OF THIS RESULT OF TEST

1. REFERENCE STANDARD INSTRUMENT:

| MASTER | MANUFACTURER | MODEL | SERIAL NO. | CERTIFICATE NO. | DUE DATE |
|----------------------------|--------------|--------|------------|-----------------|-------------|
| Electrical Safety Analyzer | Fluke | ESA612 | 2595005 | 25MD314 | 12 Feb 2026 |

2. THIS CERTIFICATION IS TRACEABLE TO :

- Technology Promotion Association (Thai-Japan)

3. THIS RESULT OF TEST WAS FOUND ACCURATE AS SHOW ON DATE AND PLACE OF TEST ONLY

Report No. 25PNH_RECPM11250830
Page 2 of 3

Preventive Maintenance Action Report

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1. Qualitative Tests

| No. | Test | Pass | Fail | Comment |
|-----|------------------------|------|------|---------|
| 1 | Chassis/Housing | / | | |
| 2 | Mount | / | | |
| 3 | Casters/Wheels/Brakes | / | | |
| 4 | AC Plug | / | | |
| 5 | Line Cord | / | | |
| 6 | Strain Reliefs | / | | |
| 7 | Circuit Breaker/Fuse | / | | |
| 8 | Cables | / | | |
| 9 | Fittings/Connectors | / | | |
| 10 | Electrodes/Transducers | / | | |
| 11 | Controls/Switches | / | | |
| 12 | Fan | / | | |
| 13 | Battery | / | | |
| 14 | Indicators/Displays | / | | |
| 15 | Self-Test | / | | |
| 16 | Time/Date Settings | / | | |
| 17 | Recorder/Printer | / | | |
| 18 | Alarms/Interlocks | / | | |
| 19 | Audible Signals | / | | |
| 20 | Labeling | / | | |
| 21 | Accessories | / | | |

2. Quantitative Tests

| No. | Test | Pass | Fail | Comment |
|-----|---|------|------|----------------|
| 1 | Grounding Resistance $\leq 0.5 \Omega$ | / | | 0.232 Ω |
| 2 | Chassis Leakage Current $\leq 500 \mu A$ | / | | 49.5 μA |
| 3 | Rate Calibration $\pm 5\%$ or 5 bpm at 60 bpm and 120 bpm | / | | |
| 4 | Rate Alarm $\pm 5\%$ or 5 bpm at 40 and 120 bpm | / | | |
| 5 | Gain $\pm 10\%$ | / | | |
| 6 | QRS Sensitivity $\geq 0.15 mV$ | / | | |
| 7 | Paper Speed $\pm 2\%$ | / | | |
| 8 | Alarm Delay $\leq 10 sec$ | / | | |

Report No. 25PNH_RECPM11250830
Page 3 of 3

Preventive Maintenance Action Report

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Protection against electrical shock Class : I , Type : BF

3. Preventive Maintenance

| No. | Test | Pass | Fail | Comment |
|-----|------------------------------------|------|------|---------|
| 1 | Clean the exterior and accessories | / | | |

Note: Electrical Safety Test : According to IEC 62353

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